

INSTRUCTIONS FOR USE

■ Cationorm®

Instructions for use

Eye drops, emulsion



Read all of this leaflet carefully before you start using this product.

Keep this leaflet; you may need to read it again.

If you have any further questions, ask your doctor or pharmacist.

If symptoms persist, consult your doctor.

1. What Cationorm® is and what it is used for

Cationorm® is an ophthalmic sterile hypotonic unpreserved eye drops emulsion in the form of a milky liquid.

Cationorm® is intended for the treatment of dry eye symptoms such as stinging, itching or burning eyes or foreign body sensation (a feeling of sand or dust) in the eyes. These symptoms may be caused by external factors, such as air conditioning, pollution, travel by plane, working on a screen, refractive surgery, contact lenses wear, etc, or by pathologies, such as meibomian gland dysfunction. Cationorm® hydrates, lubricates and protects the surface of the eye.

2. Before using Cationorm®

Do not use Cationorm® if you are allergic to any of the components of the product.

This product is not intended for treating other eye conditions. Please consult your doctor, pharmacist or optometrist if you have any questions.

If you currently use other eye drops, you should wait at least 5 minutes between the administrations of each successive eye drop. It is recommended to use Cationorm® last.


Cationorm® is compatible with all kinds of contact lenses.

3. How to use Cationorm®

USE IN THE EYE.

The recommended dose regimen is 1 drop in each eye, 1 to 4 times daily.

Instructions for use:

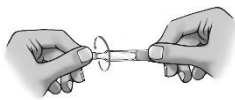

- Wash your hands.
- Take one single-dose container and shake before use.
- Twist off the top of the single-dose container.
- Tilt your head back.
- Gently pull down your lower eyelid and look up.
- Hold the single-dose container above the eye and gently squeeze 1 drop in the eye and blink several times.
- Do not touch the eye or eyelashes with the tip of the single-dose container.

Single use.

The single-dose container must be used immediately after opening and disposed of after use.

The reuse of an opened single-dose container exposes the eye to a risk of infection.

4. What are the possible side effects

In very rare cases, a short term discomfort such as: eye irritation, eye pain, foreign body sensation in the eye, eye redness, eye itching, watery eyes, burning sensation in the eye, temporarily blurred vision, eyelids inflammation or fluid retention in the eyelids can appear. These symptoms are also part of typical symptoms of dry eye disease linked to the underlying existing medical conditions in the patient's eyes suffering from dry eye.

IF YOU NOTICE ANY SIDE EFFECTS THAT ARE NOT LISTED IN THIS LEAFLET, PLEASE INFORM YOUR DOCTOR, PHARMACIST OR OPTOMETRIST.

5. How to store Cationorm®

Store the single-dose containers in their outer box at below 30°C.

Keep out of the reach and sight of children.

Do not use if the single-dose containers are damaged.

Do not use after the expiry date printed on the box.

Any opened individual single-dose container with any remaining emulsion should be discarded immediately after use.

Cationorm® contains: mineral oils, glycerol, tyloxapol, poloxamer 188, tris-hydrochloride, tromethamine, cetalkonium chloride (as cationic agent) and water for injections.

Each single-dose container of Cationorm® contains 0.4 ml of sterile emulsion.


For further information please contact your doctor/pharmacist/optometrist, Seqirus Pty. Ltd. at www.csseqirus.com (ph:1800 642 865).

For more information on Cationorm®, please visit www.cationorm.com.au or scan the QR code below.











This leaflet was prepared in November 2023.

Sponsor:
Seqirus Pty. Ltd.
Melbourne,
Victoria, Australia

 Manufacturer:
SANTEN S.A.S.
Bâtiment Genavenir IV
1 rue Pierre Fontaine
91000 EVRY COURCOURONNES
FRANCE



Key:

Symbol								
Meaning	Manufacturer	Store below 30°C	Sterile via aseptic methods	Batch Number	Expiry Date	Read the Instructions for use	Single Use Do not re-use	Do not use if the product sterile barrier system or its packaging is compromised